## JAY® Easy Visco Cushion

### Important Consumer Information

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**NOTE:** Check all parts from shipping damage. In case of damage DO NOT use. Contact Carrier/Sunrise for further instructions.

Sunrise Medical recommends that a clinician such as a doctor or therapist experienced in seating and positioning be consulted to determine if a JAY Easy Visco Cushion is appropriate. Cushions should only be installed by an authorized Sunrise Medical supplier.

### GENERAL WARNINGS!

- DO NOT install this equipment without first reading and understanding this manual.
- Keep this manual with the product for future reference.
- If you are unable to understand the Warnings, Cautions and Instructions, contact a qualified clinician or supplier - otherwise injury or damage may occur.
- Hook and Loop velcro should not come into prolonged contact with moisture as this may degrade the adhesive and lead to a failure of the velcro.
- Installing a cushion on a wheelchair may affect the center of gravity of the wheelchair and may cause the wheelchair to tip backwards, potentially resulting in injury. Always assess for the potential need for antitippers or amputee axle adapter brackets to be added to the wheelchair to help increase stability.
- Prior to prolonged sitting, any cushion should be tried for a few hours at a time while a clinician inspects your skin to ensure that red pressure spots are not developing. You should regularly check for skin redness. The clinical indicator for tissue breakdown is skin redness. If your skin develops redness, discontinue the use of the cushion immediately and see your doctor or therapist.
- The JAY Easy Visco Cushion is designed sitting comfort and postural support, remember no cushion can completely eliminate sitting pressure or prevent pressure sores.
- If you need more pressure distribution for skin protection, ask your authorised supplier for a free demonstration of any of the other Jay cushion products.
- The JAY Easy Visco Cushion is not a substitute for good skin care including; proper diet, cleanliness, and regular pressure relief.
- Allow the cushion to warm to room temperature before using. Sitting on extremely cold or hot surfaces can cause skin damage.
- Avoid sharp objects or exposure to excessive heat or open flame.
- **OBSTRUCTIONS:** Do not place any obstructions between the user and the cushion because it will reduce product effectiveness.
- **CUSHION AND COVER ORIENTATION:** Product must be used with the cushion cover facing up. If the cover is not used correctly it may reduce or eliminate the cushion's benefits and could increase risk to the skin and soft tissue.

### JAY EASY VISCO

The Easy Visco cushion is designed for high level comfort and positioning with mild pressure redistribution. It incorporates a lightweight pre-contoured foam with media/lateral thigh support and Visco elastic foam in the seat well to allow immersion of the ischia's (seat bones) and load distribution to the femurs. The area of the sacral is notched. The outer cover is moisture resistant with a no-slip bottom to keeping the cushion in place during daily use and transfers.

**Maximum user weight for this product is:**

**150 kg (330 lbs)**

### FITTING THE JAY EASY VISCO CUSHION

- Select a Jay Easy Visco Cushion that matches your hip width.
- The cushion length should extend to within 25 mm - 50 mm, (1" - 2"), from the back of the knees.
- Place the cushion to the back of the chair with the material identification tags and Visco elastic foam towards the rear.
- When fitted correctly, your hips should be to the back of the chair.
- Your ischials (seat bones) should be centered on the visco elastic foam in the seat well of the cushion.
- Proper footrest adjustment may enhance sitting comfort and help lower peak sitting pressures.
- Adjust the footrests so your legs rest firmly but comfortably on the cushion.

### NOTE:

- Do not use the wheelchair's seat width as a guide in determining the appropriate cushion width. Improper cushion fit may be detrimental to the performance of a pressure distributing cushion and its support of the body.
- Avoid using the cushion on overly stretched seat upholstery. Stretched upholstery does not provide adequate support for the cushion and improper positioning may result. Replace stretched upholstery or purchase the Jay adjustable solid seat.
- Avoid using overly stretched backrest upholstery. Stretched back upholstery does not provide adequate support for the body and improper positioning may result. Replace stretched backrest upholstery or purchase a back support product from Sunrise Medical that is appropriate for you. Contact an authorized supplier for a free demonstration of these products.

## EASY MAINTENANCE & CLEANING

Monthly cleaning and regular maintenance may help extend the life of your cushion. During cleaning, component inspection is recommended. Check the cover for tears and excessive wear.

Check the foam base and the Visco foam in the seat-well to ensure foam consistency.

### TO CLEAN COVER FIG.1.0

- Unzip the cover, "A" and Remove the cover from the base.
- Turn the cover inside out and machine wash in warm water and drip or tumble dry on low heat.
- No ironing required.

### ⚠ CAUTION!

- Do not dry clean covers or use industrial washers and dryers to clean cover.
- Do not steam autoclave.
- Do not bleach.

After washing, fit cover back over foam base. Ensure the back of the cover matches up to the back of the foam base.

### NOTE:

- If utilized in an institutional setting, write the resident's name with a permanent marker on the content label of the covers. This will help to ensure the return of the covers after washing.
- Use only Jay-designed covers in conjunction with Jay Easy Visco Cushions.

### TO CLEAN FOAM BASE FIG.1.0 - 2.0

- Remove the cover from the foam base.
- Wipe with warm water and soap.
- Wipe off with a clean cloth.
- Avoid complete submersion of the foam base in water.

**NOTE:** See Detailed Cleaning Instructions overleaf.

Fig.1.0

Fig.2.0

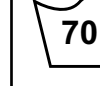
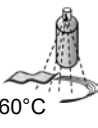
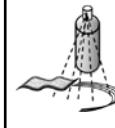
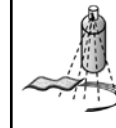
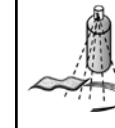
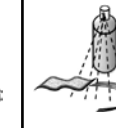
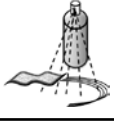
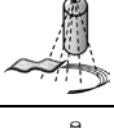
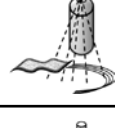
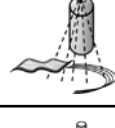
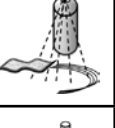
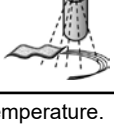
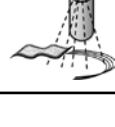
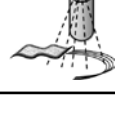
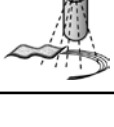
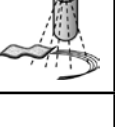
**JAY EASY VISCO CUSHION WARRANTY**

Each Jay cushion is carefully inspected and tested to provide peak performance. Every Jay cushion is guaranteed to be free from defects in materials and workmanship for a period of 24 months from the date of purchase, provided normal use. Should a defect in materials or workmanship occur within 24 months from the original date of purchase, Sunrise Medical will, at its option, repair or replace it without charge. This warranty does not apply to punctures, tears or burns, nor to the cushion's removable cover.

The removable cover is guaranteed to be free from defects in materials and workmanship for a period of six (6) months from the date of purchase, provided normal use. Should a defect in materials or workmanship occur within six (6) months from the original date of purchase, Sunrise Medical will, at its option, repair or replace it without charge.

Claims and repairs should be processed through the nearest authorized supplier. Except for express warranties made herein, all other warranties, including implied warranties of merchantability and warranties of fitness for a particular purpose are excluded.

Approved Cleaning Procedures for JAY Cushions									
Product	JAY Lite	JAY J3	J2 GS	JAY Easy Visco	JAY Soft Combi P	JAY Xtreme Active	JAY Basic	JAY Easy	JAY Balance
<b>Outer Cover (Microclimatic and Stretch)</b>									
(1). Wet Wash 70°C 									
<b>Outer Cover (Incontinent)</b>									
(2). Wet Wash 60°C 									
<b>Inner Cover</b>									
(3). Spray and Wipe disinfection  or Wet Wash 60°C 	NA	NA	NA	NA	NA		NA	NA	
<b>Cushion core (foam base)</b>									
(4). Steam autoclave up to 105 °C (1) 									
Spray and wipe disinfection 									
(5). Immersion disinfection 									
<b>Cushion elements</b>									
Roho Air element	NA			NA	NA	NA	NA	NA	
Positioning elements	NA			NA	NA		NA	NA	
Fluid pad	NA			NA	NA		NA		
<p>1. 70°C is the maximum recommended temperature.                  2. 60°C is the maximum recommended temperature.                  3. The Xtreme Active and Balance inner covers can be washed at 60°C as well as spray and wipe disinfected. Possible discoloration if washed at 60°C but the functional aspect of the covers is not diminished.                  4. An autoclave is a self locking machine that sterilizes with steam under pressure.                  5. At low temperature (&lt; 30°C)</p>									
The product warranty shall not apply to products damaged due to the failure to follow these cleaning instructions or the use of inappropriate cleaning compounds.									
<b>Recommended Disinfectants</b>									
All disinfectants which are listed and recommended by the Robert Koch Institute: <a href="http://www.rki.de/EN/Home/homepage_node.html">http://www.rki.de/EN/Home/homepage_node.html</a>									

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Sunrise Medical B.V.
Sunrise Medical HCM B.V.
Sunrise Medical S.A.S