FREETEC

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Table of contents

Preface	13
Declaration of conformity	13
Service life	13
Disposal	13
Identification plate	13
Reuse	13
Warranty	13
Parts of the product	14
Indication Contraindication	15
Scope of delivery	15
Safety instructions	15
Preparing for use	16
Attach wheels and unfold	16
Hanging the footrests	16
Length and angle adjustment of the footrest	17
Adjusting seat height (rear wheel mount)	17
Adjusting seat height (front wheel mount)	17
Adjusting the positive caster of the front wheels	17
Seat depth adjustment	18
Adjusting the backrest and cushion	18
Adjusting the height of the backrest	18
Adjusting and removing armrests	19
Adjusting and using the parking brake	19
Adjusting and using the hand brake (optional)	20
Operating the tilting aid	20
Adjusting the height of the push handles	20
Collapsing	20
Cleaning Maintenance	21
Technical data	21

Preface

Thank you for choosing a product from Drive DeVilbiss.

You will not be disappointed by the design, functionality and quality of this product.

Please read these operating instructions carefully prior to initial use. They contain important safety information and valuable tips for the proper use and care. If you have questions or need additional information, contact your Drive DeVilbiss specialist dealer who supplied your product.

These operating instructions contain all of the important information for adjustment and operation. The most current operating instructions are available as a PDF online or can be requested from customer service.

Enlarged formats for visually impaired persons can be displayed. Blind patients must be instructed by their attendant. Repairs as well as some adjustments require special technical training and must therefore be made by the medical supplies dealer.

Declaration of conformity

For the product described, DRIVE MEDICAL GMBH & Co. KG declares compliance according to the medical device regulation (EU) 2017/745.

Service life

Our company assumes a service life of five years for this product as long as the product is used as intended and all maintenance and service requirements are observed.


This service life may be significantly exceeded if the product is carefully handled, repaired, maintained and used and there are no technical limits based on the further development of science and technology.

The service life may also be significantly shortened through extreme and improper use, however. Determination of the service life by our company does not represent any additional guarantee.

Disposal

If you no longer need the product, please contact your specialist dealer or bring the product to your local disposal center.

Identification plate

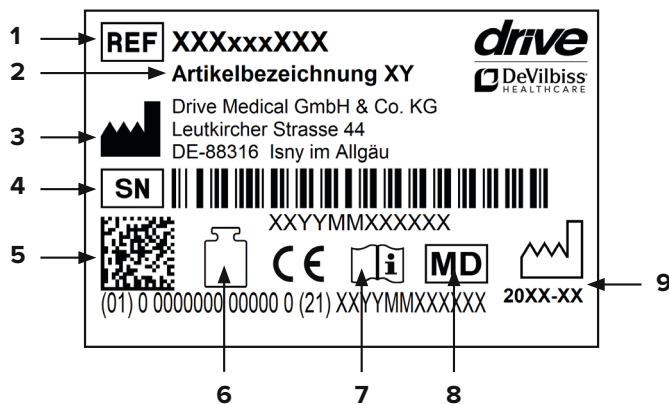1 Article number | 2 Article name | 3 Manufacturer | 4 Serial number | 5 UDI code | 6 Max. load | 7 Follow the instructions for use | 8 Medical device | 9 Production date (year-month)

Reuse

This product can be reused. Please note that all technical documents needed for safe handling must also be given to the new user.

The product must be checked by the specialist dealer, cleaned, and restored to optimum condition.

Always follow the manufacturer's instructions during preparation; these are made available on request.

Accessories are available from your specialist dealer or at www.drivedevilbiss.de

Warranty

The warranty period for the product supplied by us is 24 months from the date of purchase. Defective parts will be replaced free of charge if it can be shown that there is a defect in materials or manufacturing.

Parts subject to wear are generally not covered during the normal guarantee period unless the parts need repair or replacement which became necessary as a clear, direct consequence of a manufacturing or material error.

We reserve the right of error and modifications on the basis of technical improvements and changes in design.

EXCLUSION OF WARRANTY

Parts subject to wear include (among others): wheels, brakes, handles and armrest overlays

Parts of the product

Freetec

- | | | |
|---------------------|---|----------------------------------|
| 1. Push handle | 6. Heel strap | 10. Parking brake |
| 2. Hand brake | 7. Foot plate | 11. Quick change axle (quickpin) |
| 3. Armrest | 8. Positive and negative caster of the front wheels | 12. Hand rim wheel |
| 4. Footrest release | 9. Front swivel wheel | 13. Back cushion |
| 5. Footrests | | |

Indication | Contraindication

INDICATION:

Inability to walk or severely restricted walking ability as part of the basic need to move around in your own home and to leave your home to get some fresh air on a short walk or to reach the places usually in the surrounding area of your home where everyday business can be carried out.

The wheelchair exclusively serves people who are not able to walk and people who have problems with walking with their individual requirements when traveling themselves and traveling with others for everyday use, both inside and outside on even and solid ground. Additionally take body size, weight, physical and mental state, age, living arrangements, and environment into account in the provision of a wheelchair.

CONTRAINDICATION:

The use of the wheelchair is not suitable in cases of:

- Disorders of perception
- Severe loss of balance
- Loss of limbs to both arms
- Joint contractions/joint damage to both arms
- Inability to sit down

Scope of delivery

1 x	Wheelchair
1 x	Allen wrench
1 x	Instructions for use

Safety instructions

To ensure proper and safe use, please be sure to observe the following information:

Stability and balance

Your wheelchair is designed such that it ensures the stability you need during normal daily activities. Each of your movements in the wheelchair has an effect on the center of gravity which in the worst case can lead to tipping of the wheelchair. To increase your safety, we recommend using safety belts.

Reaching an object beyond the chair

The balance limits for reaching an object were determined through studies on a representative group of wheelchair users.

Only the arms can be extended out beyond the wheelchair seat. The body and the head should remain within the limits of the seat.

Do not lean forward

Do not lean your chest over the armrest. To reach an object in front of you, you must lean forward and bend downwards. For this reason, you must use the steering wheels as an aid (by facing them forwards) in order to attain stability and balance. Accurate orientation of the wheels is important for your safety.

Do not lean back

Position the wheelchair as close as possible to the desired object so that you can easily reach it with outstretched arms while you remain seated in the chair in a normal position. Do not lean back too far in any case since you can otherwise cause the chair to tip over.

Lateral transfer

This transfer can be performed alone, provided you have appropriate mobility and muscle tone. Position the wheelchair as close as possible to the place where you wish to sit such that the steering wheels are pointing forwards. Secure the parking brakes of the rear wheels.

Use a transfer board, if possible.

Forward transfer

If you are able to stand up, you can transfer yourself forwards to another seat. If this movement is difficult for you, we recommend obtaining assistance from an attendant or caregiver to ensure a transfer that is as easy and safe as possible.

Secure both brakes and ensure that both steering wheels are pointing forwards. In doing so, be aware that you should not under any circumstances support yourself on the brakes during the transfer. Then grasp the top of the armrests, support yourself on your arms in order to easily get up and the attendant can assist you in completing the transfer. Position yourself as near as possible to the place where you wish to sit.

During the transfer, position yourself as far back in the seat as possible to prevent the seat cushion from being damaged or the wheelchair from tipping forward.

Secure the two brakes; the brake handles should never be used for support during transfers. Never support yourself on the foot plates when getting up from the wheelchair.

If you are able to stand more or less upright and if you have sufficient upper body strength and mobility, you can transfer yourself forwards to another seat. Fold up the foot plate and push the foot support/leg support to the side to clear the way, bend your body forwards while you support yourself on the two armrests and stand up. Then push your body in the direction of the place where you wish to sit while distributing your weight on your arms and hands.

Overcoming obstacles with an attendant

Getting up onto the sidewalk

The attendant positions the wheelchair facing forward in front of the sidewalk. He/she then tilts the wheelchair backwards towards him-/herself until the steering wheels reach up onto the sidewalk; he/she pushes the wheelchair forward until the rear wheels are at the edge of the curb and pushes the wheelchair further until the rear wheels roll over the edge of the curb.


Getting down off the sidewalk

The attendant positions the wheelchair facing forward with the steering wheels at the edge of the curb and tilts it backwards towards him-/herself. Then he/she carefully pushes the wheelchair forwards until the rear wheels touch the street, then he/she gradually lowers the steering wheels to the ground.

Getting over stairs

Since it is difficult to get over stairs, we recommend getting assistance from two attendants, one in front of the wheelchair and one behind the wheelchair at the push handles. The attendant who is positioned behind the wheelchair tilts it backwards towards him-/herself while holding the handles firmly. The attendant who is positioned in front of the wheelchair holds the front lateral frame firmly and lifts the chair over the stairs while the other person places one foot on the next stair and repeats the step.

Do not lower the steering wheels of the wheelchair until the attendant positioned in front of the wheelchair has crossed the last stair. To go down stairs, the same procedure as described above should be followed, in reverse order.

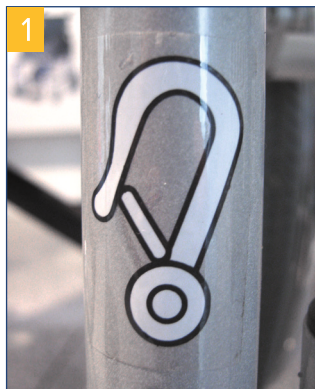1 Lifting the wheelchair with hoists

If the wheelchair needs to be lifted with a gantry crane, for example, to overcome obstacles, only the 4 points on the frame of the wheelchair which are identified with a hook sticker (see photo, left) may be used as lifting points.

Do not attempt to lift the wheelchair by grasping any removable parts (for example, the armrests, leg supports or foot supports). Avoid using an escalator.

Traveling on inclines by independent wheelchair users

We recommend avoiding ramps with an incline of more than 9°. In the case of larger inclines, there is a risk of tipping over if the wheels spin or when approaching an incline at an oblique angle.

Going up ramps

Lean your upper body forward and operate the hand rims with rapid pushes to reach an appropriate speed. If you want to stop to rest along the way, the two brakes must be operated simultaneously.

Going down ramps

Lean your upper body backwards and use the hand rims which slide through your hands to control the speed and direction.

Avoid sudden direction changes, never attempt to ascend or descend a ramp diagonally, and do not attempt to turn the wheelchair when ascending. The wheelchair is not suitable as a seat for driving a vehicle.

Take note of the permitted maximum weight of the user. This information can be found in the technical data in these instructions for use as well as on the identification plate.

Protect the product from direct sunlight

Product parts can become hot and cause skin injuries if touched.

Every serious incident which occurs in connection with the product must be reported to the manufacturer and the competent authority of the member state in which the user and/or patient is located.

Preparing for use

Attach wheels and unfold

Insert the hand rim wheels into the hole provided in the Varioblock by pressing the release button in the center of the hub. Ensure that the wheel is properly locked after insertion; check this by pulling outwards (**photo 2+3**). Now unfold the wheelchair by pressing on the seat surface until it noticeably clicks into place.

Hanging the footrests

Hang both footrests provided as shown and swivel them inwards until they audibly click into place (**photo 4+5**). To release the footrests, press the lever inwards and swivel the footrests to the side (**photo 6**).


Never swivel the footrests inwards!

Length and angle adjustment of the footrest

Length adjustment (photo 7)

Use the enclosed Allen wrench to unscrew and remove the screw on the footrest and adjust the length as desired by shifting up or down. Check where the threaded hole is and reinsert the locking screw.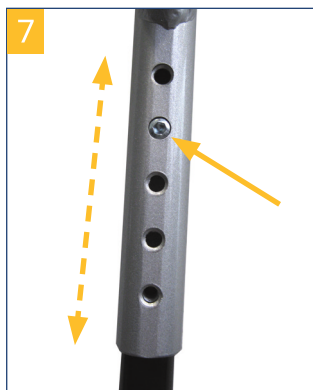

Retighten the screw!

Angle adjustment (photo 8)

Use the enclosed Allen wrench to release the screw on the footrest until it can be pulled out of the toothed wheel inwards. Now rotate the footrest into the desired position and press it back into the toothed wheel.


Retighten the screw!

Adjusting seat height (rear wheel mount)

The height of the seat at the rear can be adjusted in increments. To adjust it, remove both hand rim wheels by pressing the release button (see section "Attach wheels"). Unscrew the nut of the wheel mount and move it into the desired hole.


Replace the nut with the locking washer and tighten it (photo 9).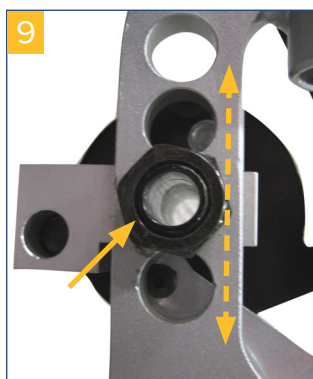

Ensure that the wheel axle is properly tightened!

Adjusting seat height (front wheel mount)

The height of the seat can be adjusted in increments and is correspondingly adjusted in the desired inclination of the seat surface. To adjust, unscrew the wheel axle screw and pull it out.
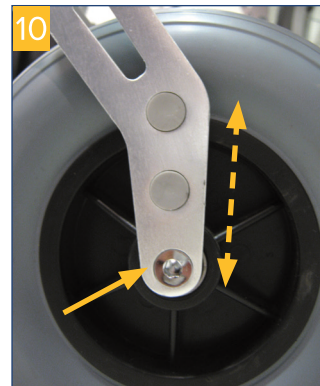


The plugs of the corresponding holes can be easily exchanged. Now move the wheel in the desired position and reinsert the wheel axle screw and tighten it (photo 10).

Ensure that the wheel axle is properly tightened!

Adjusting the positive caster of the front wheels


To adjust or shift the positive and negative caster of the front wheels, use the enclosed Allen wrenches to unscrew both lower locking screws (set screws - photo 11). Then the 2 screws on the eccentric block need to be unscrewed. The negative or positive caster can be adjusted by rotating right or left (photo 12).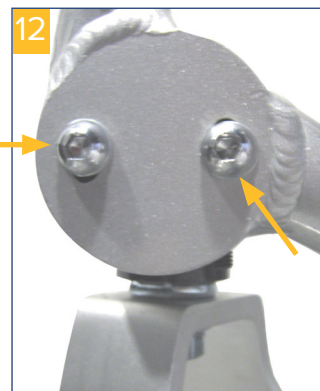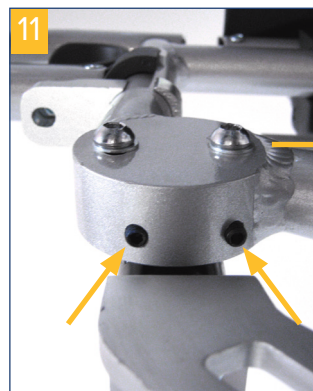



This adjustment should be performed only by the specialist dealer!

Seat depth adjustment

Upon delivery, the seat depth of all models is set at 43 cm. This seat depth can be adjusted in a total of four increments up to 50 cm. Before adjusting the seat depth, open both Velcro fasteners on the seat cushion (**photo 13**) and fold the front part of the cushion forwards (**photo 14**).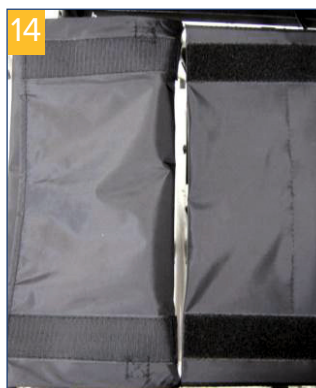

To adjust the seat depth, remove both screws on the upper longitudinal tube on both sides (**photo 15**) as well as the four screws below on the wheel mount fork, left and right (**photo 16**).

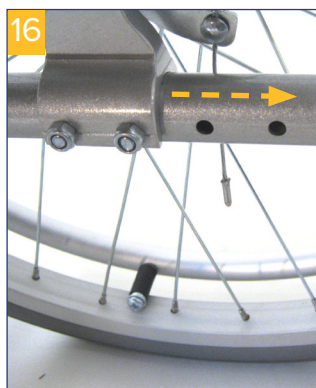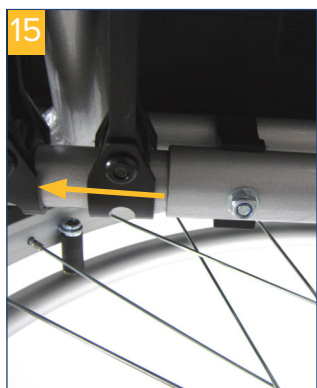Now you can pull the frame forward to the next hole (a total of three times) and reinsert the screws.

Ensure that all screws are reinserted and tightened!

Fold the front seat cushion back again and refasten the Velcro fasteners.

Adjusting the backrest and cushion

The degree of firmness of the backrest can be individually and continuously adjusted using the 5 Velcro straps (loosen or tighten) (**photo 17**).

To attach the back cushion, place it as shown (**photo 18**) on the wheelchair and fold it over in successive parts.

Place part 1 on the inside of the backrest, part 2 on the back side, and smooth part 3 from behind under the seat surface.

The cushion is secured automatically by the integrated Velcro straps (**photo 19+20**).


Adjusting the height of the backrest

The height of the backrest can be adjusted in increments.

Note: When the backrest is adjusted, the height of the push handles is also automatically adjusted.

To adjust the backrest, remove the left and right nuts under the guide tube of the backrest (**photo 21**).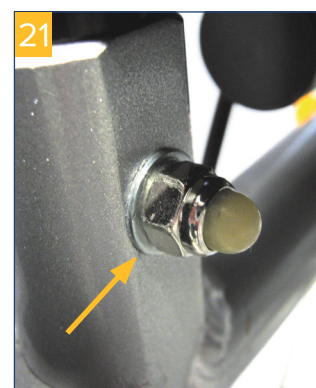

Pull the screws out. Now adjust the desired height by pulling or pushing on the respective push handle.

Reinsert the screws, put on the knurled nuts and tighten them.

Always ensure that both sides are adjusted identically.

Adjusting and removing armrests


The armrests are already assembled on the wheelchair upon delivery. The armrest overlay can be adjusted as a short or long arm support as needed (**photo 22+23**)

To set, press the spring button in the middle on the outside of the armrest overlay (**photo 24**); this can now be pushed into the desired position.


The spring button must lock back into position after adjustment. Select the closest position, if necessary.

To swivel the armrest backwards, engage the release lever at the front of the armrest and hold it down while simultaneously pulling the armrest up until the armrest is free. Now it can be swiveled backwards (**photo 25+26**).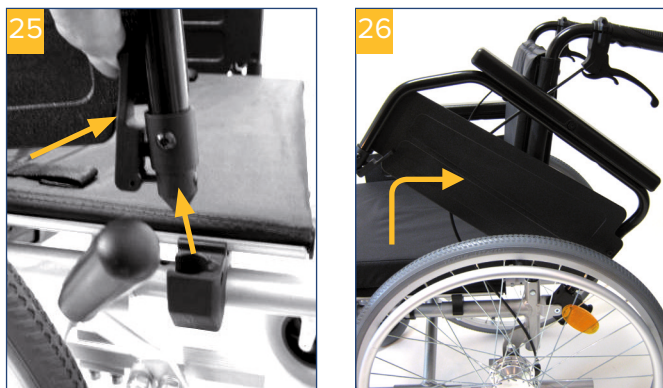

If you would like to completely remove the armrests, release them in the front as described above and lift them slightly so that they are free.


Now release the entire armrest by pulling and pressing the rear release button (**photo 27**). This can now be removed by lifting off (**photo 28**).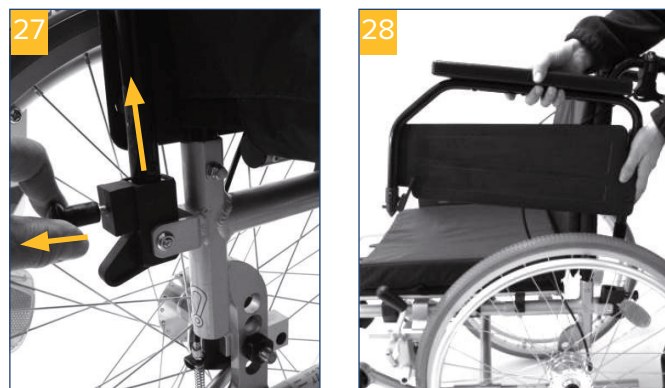

Adjusting and using the parking brake

Upon delivery, the parking brake is correctly adjusted according to the configuration status. If you have made changes to the wheel position, the brake adjustment may need to be changed.

To do this, remove both screws on the underside of the respective brake (**photo 29**) and push the complete unit onto the guide such that there is a distance of 10 mm between the brake bolt and the tire (**photo 30**). Retighten both screws and the parking brake is now optimally adjusted.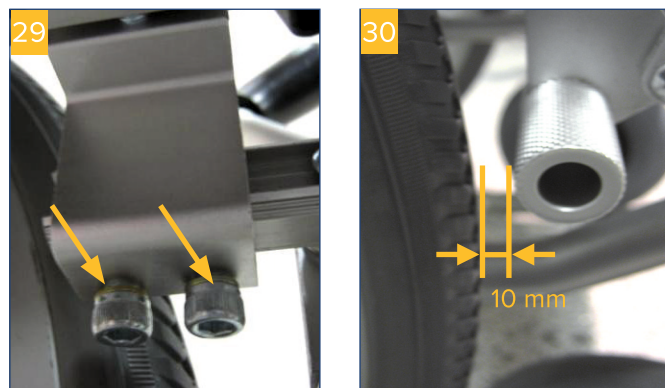


Ensure that all screws are reinserted and tightened!

To use the parking brake, press the brake lever firmly

from the front until it noticeably clicks into place – the parking brake is now engaged (**photo 31**).


To release, pull the brake lever towards you and the parking brake is unlocked (**photo 32**).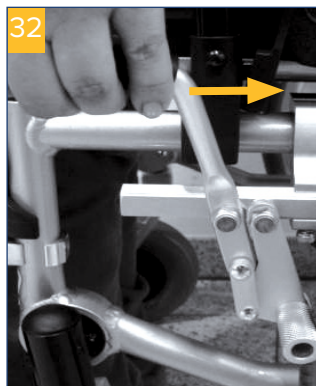

Always ensure that the parking brake is locked by attempting to move the wheelchair.

Adjusting and using the hand brake (optional)

Upon delivery, the optional hand brake is correctly adjusted for the attendant. If braking is insufficient, this can be increased by unscrewing the adjusting screw and decreased by screwing in the adjusting screw (**photo 33**).

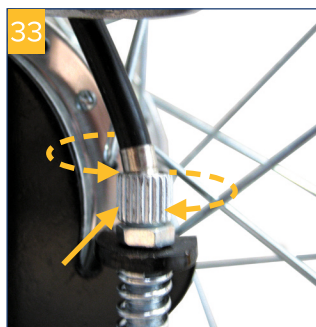For adjustment, first remove the locking nut and then perform the corresponding adjustment; thereafter retighten the locking nut and in doing so, use pliers to hold the adjusting screw. To brake the wheelchair by the attendant, pull up both brake handles at the hand grip.

To apply the brakes (such as for parking), you can lock the brake handle in the braking position by simply pushing the locking mechanism forwards with your finger until it snaps into place; to unlock, push the locking mechanism back (**photo 34+35**).

If you need to step away from the wheelchair while it is occupied, it is essential to additionally apply the parking brake.

Operating the tilting aid

To maneuver around curbstones and over thresholds step firmly (with the left or right foot) on the tilt aid while simultaneously pushing down the handles. This ensures that lifting the front of the wheelchair is easier. Ensure that you are standing securely (**photo 36**).

Adjusting the height of the push handles

To adjust the height of the push handles, remove the clamping screw and the height can then be continuously adjusted (**photo 37+38**).

Tighten the clamping screw!

Collapsing

For easier transport or for stowing in motor vehicles, the wheelchair can be collapsed with just a single maneuver. To do this, first remove the optional seat cushion as well as the footrests, then grasp both looped handles located at the side of the seat surface cushion and pull the wheelchair upwards (**photo 39+40**).

Cleaning | Maintenance

- Clean regularly with a moist cloth!
- Do not use any aggressive cleaning agents!
- Do not use abrasive cleaners!
- Do not use any high-dose chlorine and vinegar-based cleaners!
- To disinfect, a spray disinfectant can be used such as Descosept AF or Bacillol AF.
- Check the screws regularly and tighten, if necessary.
- The product should not be exposed to temperatures above 60°C for long periods of time (for example, use in a sauna, etc.).
- In the event of defects, contact your specialist dealer from whom you purchased this product.
- Check the condition of the tires regularly. They are "fail-safe," meaning that you do not have to check air pressure or fill them with air. Always check the tires for tears or other damage, however.

Technical data

Product	Freetec lightweight wheelchair							
Seat width	38 cm	41 cm	43 cm	45 cm	48 cm	50 cm	52 cm	54 cm
With drum brake	900550100	900550200	900550300	900550400	900550500	900550600	900550700	900550800
Total width with drumbrake	58,5 cm	61,5 cm	63,5 cm	65,5 cm	68,5 cm	70,5 cm	72,5 cm	75,5 cm
Width, folded	30,5 cm	30,5 cm	30,5 cm	30,5 cm	30,5 cm	30,5 cm	30,5 cm	30,5 cm
Weight	18 kg	18,3 kg	18,4 kg	18,5 kg	18,6 kg	18,8 kg	19 kg	19,2 kg

Product	Freetec lightweight wheelchair							
Seat width	38 cm	41 cm	43 cm	45 cm	48 cm	50 cm	52 cm	54 cm
Without drum brake	900500100	900500200	900500300	900500400	900500500	900500600	900500700	900500800
Total width without drum brake	56 cm	59 cm	61 cm	63 cm	66 cm	68 cm	70 cm	73 cm
Width, folded	28 cm	28 cm	28 cm	28 cm	28 cm	28 cm	28 cm	28 cm
Weight	16 kg	16,3 kg	16,4 kg	16,5 kg	16,6 kg	16,8 kg	17 kg	17,2 kg

General technical data	
HMV (medical aids register) no.	18.50.02.2112
Total length with/without footrests	103 cm / 80 cm
Overall height	92 - 99,5 cm (additional 11 cm continuous handle height adjustment)
Seat depth	43 - 50 cm
Seat height	45 - 50 cm
Height of backrest	38,5 - 46 cm
Footrest length	34,5 - 48,5 cm
Max. load	125 kg

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Drive Medical GmbH & Co. KG